access

en	Plastic holders and foam discs
fr	Support en plastique et disques endo
it	Supporto in plastica e spugne
es	Soporte de plástico y discos esponjas
pt	Suporte de plástico e discos de espuma
de	Halterung aus Kunststoff und Schaumstoffscheiben
nl	Plastic steunen en Schuimstofschijven

Instruction for use

en

WARNING NOTICE: THE PRODUCT IS NOT SUPPLIED STERILE. STERILIZE IT BEFORE USE.

Restriction on treatments:

- The plastic holders are reusable after sterilisation
- Foam discs are for single use.

Use:

ENDO-CLEAN HOLDER
OVAL ACCESS

Plastic holder for cleaning and storing endodontic instruments during treatment.
- For use with foam discs and swabs.
- Can be filled with disinfectant solution.
- Autoclavable at 134°C.

ENDO-CLEAN ACCESS
DISCS
(x 25) diameter 6,5 cm

Single-use foam discs for use with disinfectant solution.

Method of use:

Clean the instruments (files or reamers) by a rotating movement, guided by a hand inside the sponge. The impurities left in the endodontic instruments will be removed by the porosity of the sponges.

Setting up the support:

Insert the sponge between the two interlocking plastic parts.

Disinfection and sterilisation:

Before use, sterilise in an autoclave at 134°C for 18 minutes. The product may change colour slightly, but this will not affect its use or performance.

Maintenance:

No specific requirements.

Disposal:

Dispose of the product according to the correct disposal procedures.

Inspection and function test:

Visual check of any damage or deformation of the product.

Storage:

No specific requirements.

The instructions listed above have been validated by the medical device manufacturer as suitable for preparing the device for use. It is the responsibility of the user to ensure that the preparation process, equipment, materials used, and personnel are adequate to achieve the desired results.

Any deviation from the recommended instructions must be carefully analysed to determine the effectiveness and possible negative consequences.

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