access

en	Sequencer	pt	Sequenciador
fr	Sequenceur	de	Sequencer
it	Sequenziatore	nl	Sequencer
es	Secuenciador		

Instruction for use	en
WARNING NOTICE: THE PRODUCT IS NOT SUPPLIED STERILE. STERILIZE IT BEFORE USE.	

This product is reusable after sterilisation.

Use:

Endodontic sequencer with 8 perforations and 2 graduated scales, providing sterilisation, storage and support for a sequence of endodontic Instruments.

- Closed, the ACCESS sequencer serves as a support for the sterilization and storage of endodontic Instruments.
- Open, the sequencer serves as a support for easy handling of the instruments.

Disinfection and sterilisation:

Before use, sterilise in an autoclave at 134°C for 18 minutes. The product may change colour slightly, but this will not affect its use or performance.

Maintenance:

No specific requirements.

Disposal:

Dispose of the product according to the correct disposal procedures.

Inspection and function test:

Visual check of any damage or deformation of the product.

Storage:

No specific requirements.

The instructions listed above have been validated by the medical device manufacturer as suitable for preparing the device for use.

It is the responsibility of the user to ensure that the preparation process, equipment, materials used, and personnel are adequate to achieve the desired results.

Any deviation from the recommended instructions must be carefully analysed to determine the effectiveness and possible negative consequences.

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